


KIATO Safety Scalpels help to minimize the risk of surgical blade injuries that occur before use, during passing, between steps of the procedure, after use and during disposal. The Safety Scalpel is designed to be used single handedly.

### 1. Ship/Pass position:



The retractable sheath securely covers the blade fully. The latch must be released to retract the sheath backwards to the Use Position. The sheath does not retract backwards without activation.

### 2. Use position:

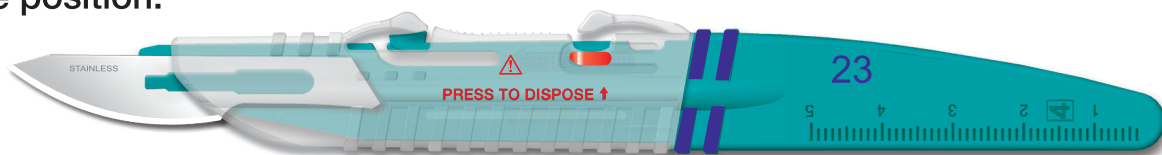

The Surgical Blade is exposed and ready for use. Audible click confirms sheath is fully retracted.




Packed in easy peel rigid blister pouches. 10 scalpels per unit box. Boxes are colour coded for easy identification. 10 boxes per case. 10 case per Master Shipper.

## Directions for use


### To Use:

Press Release button marked On to depress latch on handle and retract sheath fully until it clicks. The blade is now exposed and the sheath is locked in the "USE" position

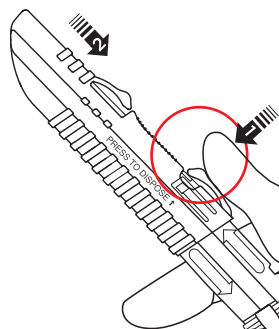


### To Pass:

Press Release button marked Off and slide sheath forward to initial position until it locks in the pass position.

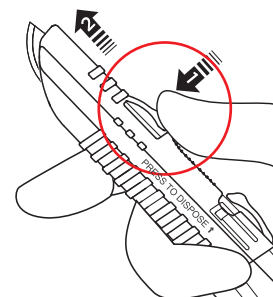


### To Dispose

Slide Sheath to "pass position". Push dispose button inwards into handle until it clicks & locks.

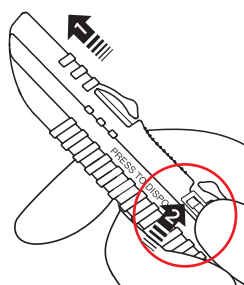


### Step 1

Dispose in an approved sharps container.

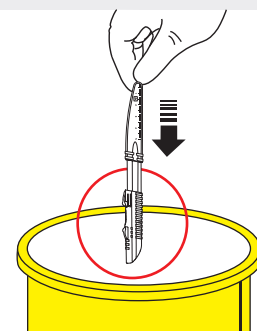

### Step 2

- Premium Swedish Stainless Steel.
- Colour coded handle with Size No clearly printed
- Durable sharpness.
- Clearly printed instructions.
- Cap Safely locked in ship & pass position.
- Permanent Dispose button

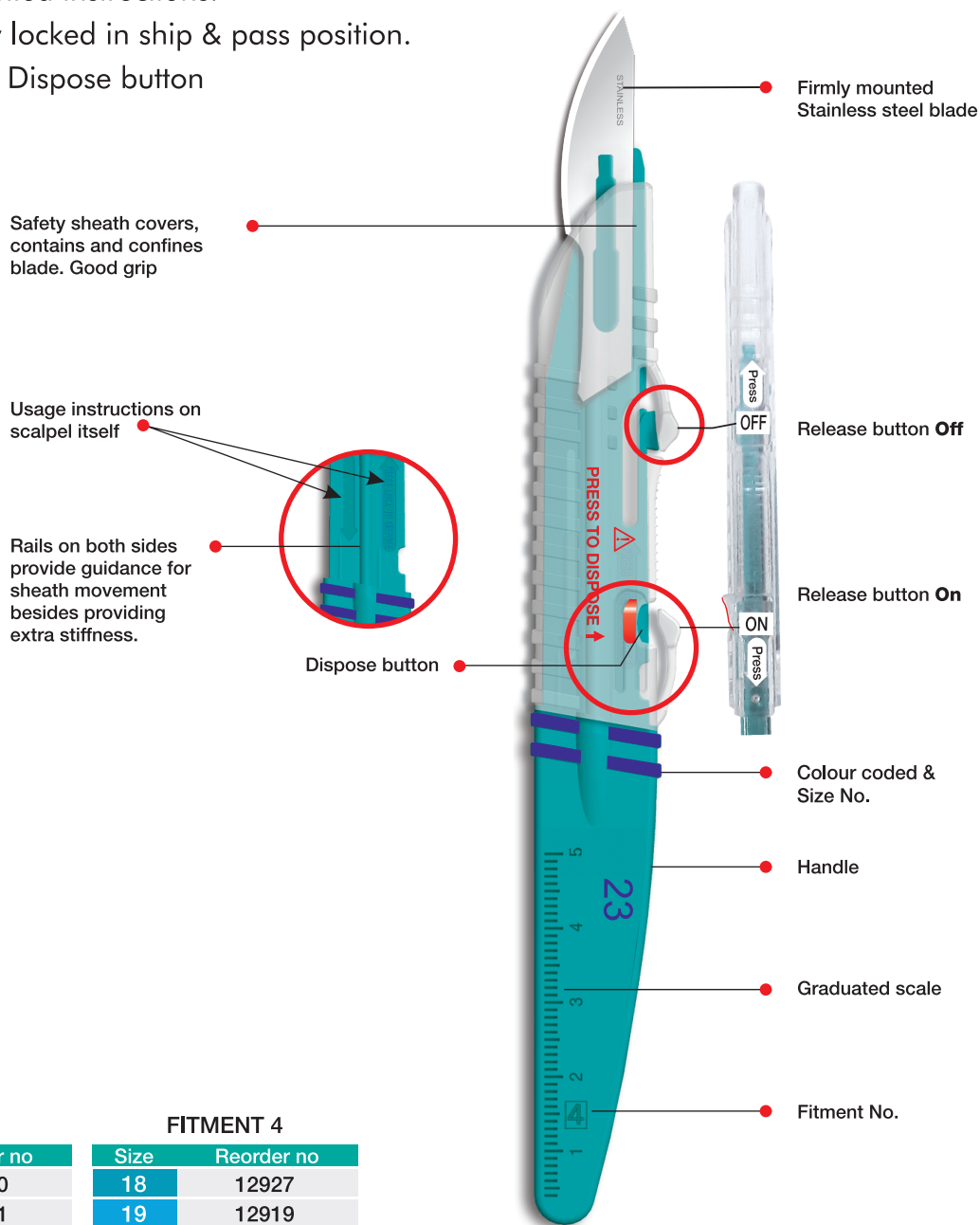

### FITMENT 3

Size	Reorder no
10	12910
11	12911
11P	12931
12	12912
12D	12925
13	12913
15	12915
15C	12926

### FITMENT 4

Size	Reorder no
18	12927
19	12919
20	12920
21	12921
22	12922
23	12923
24	12924
25	12928



MDSS GmbH  
Schiffgraben 41  
30175 Hannover  
Germany



**Aditya Dispomed Products Pvt. Ltd.**  
Plot No. 19 Sector 6 IMT Manesar  
Gurgaon Haryana 122050 India  
Tel. +91 124 47 64 900  
Fax +91 124 47 64 911  
info@adityadispomed.com  
www.adityadispomed.com

The smart choice for sharp blades

A subsidiary of Kehr Surgical Pvt. Ltd.