

Safety Scalpel

Sterile Disposable Safety Scalpel with Retractable Sheath

One-handed blade control with permanent post-use lock — designed to help reduce sharps injuries.

Single use • Latex free • Stainless steel blade

Do not autoclave

KIATO Plus® safety scalpels are designed to help reduce the risk of sharps injuries occurring **before use, during passing, after use, and during disposal.**

The scalpel features a **retractable safety sheath, one-handed ambidextrous operation**, and a **Permanent Lock Dispose Button** that prevents blade re-exposure after use. Audible and tactile feedback **confirm** correct activation at each stage, supporting safe handling in demanding clinical environments.


KIATO Plus® safety scalpels help healthcare facilities comply with **European Directive 2010/32/EU** on the prevention of sharps injuries in the hospital and healthcare sector.

1. Ship / Pass position:



The safety sheath fully covers the blade. The sheath remains locked and **cannot retract without activation**, ensuring protection during transport and passing.

2. Use position:

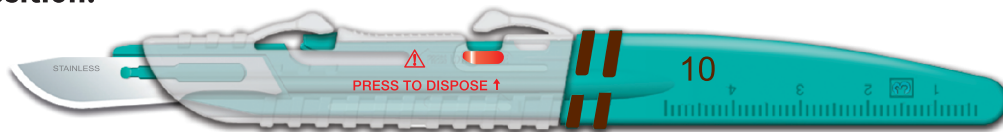


The blade is exposed and ready for use. An **audible click** confirms the sheath is fully retracted and locked in the USE position.

Directions for Use


To Use

- Press the **release button** on the handle
- Retract the safety sheath fully
- Listen for the **audible click** confirming the blade is locked in the USE position

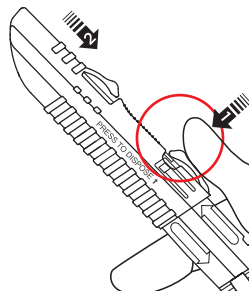


To Pass

- Press the **release button**
- Slide the safety sheath forward to the initial position
- Sheath locks securely in the PASS position

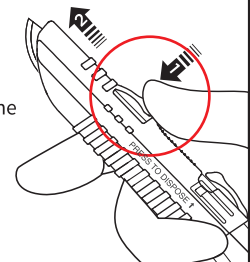

To Dispose


- Slide the sheath to the PASS position
- Press the **Permanent Lock Dispose Button** inward until it clicks
- Dispose immediately in an approved sharps container

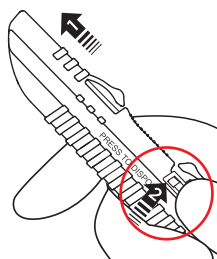

Step 1

- Once the **Permanent Lock Dispose Button** is activated, the blade cannot be re-exposed.

Dispose in an approved sharps container.

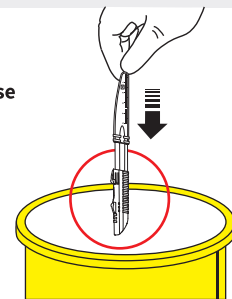

Step 2

KIATO⁺ PLUS STERILE SAFETY SCALPEL
with Retractable Sheath Stainless Steel Blade

Contents Sterile in unopened, undamaged package.

Pat. No. 7,087,067
AF-314/5

MDI, ISO 9001, ISO 13485, CE, EN 12311, EN 12312-1, EN 12312-2, EN 12312-3, EN 12312-4, EN 12312-5, EN 12312-6, EN 12312-7, EN 12312-8, EN 12312-9, EN 12312-10, EN 12312-11, EN 12312-12, EN 12312-13, EN 12312-14, EN 12312-15, EN 12312-16, EN 12312-17, EN 12312-18, EN 12312-19, EN 12312-20, EN 12312-21, EN 12312-22, EN 12312-23, EN 12312-24, EN 12312-25, EN 12312-26, EN 12312-27, EN 12312-28, EN 12312-29, EN 12312-30, EN 12312-31, EN 12312-32, EN 12312-33, EN 12312-34, EN 12312-35, EN 12312-36, EN 12312-37, EN 12312-38, EN 12312-39, EN 12312-40, EN 12312-41, EN 12312-42, EN 12312-43, EN 12312-44, EN 12312-45, EN 12312-46, EN 12312-47, EN 12312-48, EN 12312-49, EN 12312-50, EN 12312-51, EN 12312-52, EN 12312-53, EN 12312-54, EN 12312-55, EN 12312-56, EN 12312-57, EN 12312-58, EN 12312-59, EN 12312-60, EN 12312-61, EN 12312-62, EN 12312-63, EN 12312-64, EN 12312-65, EN 12312-66, EN 12312-67, EN 12312-68, EN 12312-69, EN 12312-70, EN 12312-71, EN 12312-72, EN 12312-73, EN 12312-74, EN 12312-75, EN 12312-76, EN 12312-77, EN 12312-78, EN 12312-79, EN 12312-80, EN 12312-81, EN 12312-82, EN 12312-83, EN 12312-84, EN 12312-85, EN 12312-86, EN 12312-87, EN 12312-88, EN 12312-89, EN 12312-90, EN 12312-91, EN 12312-92, EN 12312-93, EN 12312-94, EN 12312-95, EN 12312-96, EN 12312-97, EN 12312-98, EN 12312-99, EN 12312-100, EN 12312-101, EN 12312-102, EN 12312-103, EN 12312-104, EN 12312-105, EN 12312-106, EN 12312-107, EN 12312-108, EN 12312-109, EN 12312-110, EN 12312-111, EN 12312-112, EN 12312-113, EN 12312-114, EN 12312-115, EN 12312-116, EN 12312-117, EN 12312-118, EN 12312-119, EN 12312-120, EN 12312-121, EN 12312-122, EN 12312-123, EN 12312-124, EN 12312-125, EN 12312-126, EN 12312-127, EN 12312-128, EN 12312-129, EN 12312-130, EN 12312-131, EN 12312-132, EN 12312-133, EN 12312-134, EN 12312-135, EN 12312-136, EN 12312-137, EN 12312-138, EN 12312-139, EN 12312-140, EN 12312-141, EN 12312-142, EN 12312-143, EN 12312-144, EN 12312-145, EN 12312-146, EN 12312-147, EN 12312-148, EN 12312-149, EN 12312-150, EN 12312-151, EN 12312-152, EN 12312-153, EN 12312-154, EN 12312-155, EN 12312-156, EN 12312-157, EN 12312-158, EN 12312-159, EN 12312-160, EN 12312-161, EN 12312-162, EN 12312-163, EN 12312-164, EN 12312-165, EN 12312-166, EN 12312-167, EN 12312-168, EN 12312-169, EN 12312-170, EN 12312-171, EN 12312-172, EN 12312-173, EN 12312-174, EN 12312-175, EN 12312-176, EN 12312-177, EN 12312-178, EN 12312-179, EN 12312-180, EN 12312-181, EN 12312-182, EN 12312-183, EN 12312-184, EN 12312-185, EN 12312-186, EN 12312-187, EN 12312-188, EN 12312-189, EN 12312-190, EN 12312-191, EN 12312-192, EN 12312-193, EN 12312-194, EN 12312-195, EN 12312-196, EN 12312-197, EN 12312-198, EN 12312-199, EN 12312-200, EN 12312-201, EN 12312-202, EN 12312-203, EN 12312-204, EN 12312-205, EN 12312-206, EN 12312-207, EN 12312-208, EN 12312-209, EN 12312-210, EN 12312-211, EN 12312-212, EN 12312-213, EN 12312-214, EN 12312-215, EN 12312-216, EN 12312-217, EN 12312-218, EN 12312-219, EN 12312-220, EN 12312-221, EN 12312-222, EN 12312-223, EN 12312-224, EN 12312-225, EN 12312-226, EN 12312-227, EN 12312-228, EN 12312-229, EN 12312-230, EN 12312-231, EN 12312-232, EN 12312-233, EN 12312-234, EN 12312-235, EN 12312-236, EN 12312-237, EN 12312-238, EN 12312-239, EN 12312-240, EN 12312-241, EN 12312-242, EN 12312-243, EN 12312-244, EN 12312-245, EN 12312-246, EN 12312-247, EN 12312-248, EN 12312-249, EN 12312-250, EN 12312-251, EN 12312-252, EN 12312-253, EN 12312-254, EN 12312-255, EN 12312-256, EN 12312-257, EN 12312-258, EN 12312-259, EN 12312-260, EN 12312-261, EN 12312-262, EN 12312-263, EN 12312-264, EN 12312-265, EN 12312-266, EN 12312-267, EN 12312-268, EN 12312-269, EN 12312-270, EN 12312-271, EN 12312-272, EN 12312-273, EN 12312-274, EN 12312-275, EN 12312-276, EN 12312-277, EN 12312-278, EN 12312-279, EN 12312-280, EN 12312-281, EN 12312-282, EN 12312-283, EN 12312-284, EN 12312-285, EN 12312-286, EN 12312-287, EN 12312-288, EN 12312-289, EN 12312-290, EN 12312-291, EN 12312-292, EN 12312-293, EN 12312-294, EN 12312-295, EN 12312-296, EN 12312-297, EN 12312-298, EN 12312-299, EN 12312-300, EN 12312-301, EN 12312-302, EN 12312-303, EN 12312-304, EN 12312-305, EN 12312-306, EN 12312-307, EN 12312-308, EN 12312-309, EN 12312-310, EN 12312-311, EN 12312-312, EN 12312-313, EN 12312-314, EN 12312-315, EN 12312-316, EN 12312-317, EN 12312-318, EN 12312-319, EN 12312-320, EN 12312-321, EN 12312-322, EN 12312-323, EN 12312-324, EN 12312-325, EN 12312-326, EN 12312-327, EN 12312-328, EN 12312-329, EN 12312-330, EN 12312-331, EN 12312-332, EN 12312-333, EN 12312-334, EN 12312-335, EN 12312-336, EN 12312-337, EN 12312-338, EN 12312-339, EN 12312-340, EN 12312-341, EN 12312-342, EN 12312-343, EN 12312-344, EN 12312-345, EN 12312-346, EN 12312-347, EN 12312-348, EN 12312-349, EN 12312-350, EN 12312-351, EN 12312-352, EN 12312-353, EN 12312-354, EN 12312-355, EN 12312-356, EN 12312-357, EN 12312-358, EN 12312-359, EN 12312-360, EN 12312-361, EN 12312-362, EN 12312-363, EN 12312-364, EN 12312-365, EN 12312-366, EN 12312-367, EN 12312-368, EN 12312-369, EN 12312-370, EN 12312-371, EN 12312-372, EN 12312-373, EN 12312-374, EN 12312-375, EN 12312-376, EN 12312-377, EN 12312-378, EN 12312-379, EN 12312-380, EN 12312-381, EN 12312-382, EN 12312-383, EN 12312-384, EN 12312-385, EN 12312-386, EN 12312-387, EN 12312-388, EN 12312-389, EN 12312-390, EN 12312-391, EN 12312-392, EN 12312-393, EN 12312-394, EN 12312-395, EN 12312-396, EN 12312-397, EN 12312-398, EN 12312-399, EN 12312-400, EN 12312-401, EN 12312-402, EN 12312-403, EN 12312-404, EN 12312-405, EN 12312-406, EN 12312-407, EN 12312-408, EN 12312-409, EN 12312-410, EN 12312-411, EN 12312-412, EN 12312-413, EN 12312-414, EN 12312-415, EN 12312-416, EN 12312-417, EN 12312-418, EN 12312-419, EN 12312-420, EN 12312-421, EN 12312-422, EN 12312-423, EN 12312-424, EN 12312-425, EN 12312-426, EN 12312-427, EN 12312-428, EN 12312-429, EN 12312-430, EN 12312-431, EN 12312-432, EN 12312-433, EN 12312-434, EN 12312-435, EN 12312-436, EN 12312-437, EN 12312-438, EN 12312-439, EN 12312-440, EN 12312-441, EN 12312-442, EN 12312-443, EN 12312-444, EN 12312-445, EN 12312-446, EN 12312-447, EN 12312-448, EN 12312-449, EN 12312-450, EN 12312-451, EN 12312-452, EN 12312-453, EN 12312-454, EN 12312-455, EN 12312-456, EN 12312-457, EN 12312-458, EN 12312-459, EN 12312-460, EN 12312-461, EN 12312-462, EN 12312-463, EN 12312-464, EN 12312-465, EN 12312-466, EN 12312-467, EN 12312-468, EN 12312-469, EN 12312-470, EN 12312-471, EN 12312-472, EN 12312-473, EN 12312-474, EN 12312-475, EN 12312-476, EN 12312-477, EN 12312-478, EN 12312-479, EN 12312-480, EN 12312-481, EN 12312-482, EN 12312-483, EN 12312-484, EN 12312-485, EN 12312-486, EN 12312-487, EN 12312-488, EN 12312-489, EN 12312-490, EN 12312-491, EN 12312-492, EN 12312-493, EN 12312-494, EN 12312-495, EN 12312-496, EN 12312-497, EN 12312-498, EN 12312-499, EN 12312-500, EN 12312-501, EN 12312-502, EN 12312-503, EN 12312-504, EN 12312-505, EN 12312-506, EN 12312-507, EN 12312-508, EN 12312-509, EN 12312-510, EN 12312-511, EN 12312-512, EN 12312-513, EN 12312-514, EN 12312-515, EN 12312-516, EN 12312-517, EN 12312-518, EN 12312-519, EN 12312-520, EN 12312-521, EN 12312-522, EN 12312-523, EN 12312-524, EN 12312-525, EN 12312-526, EN 12312-527, EN 12312-528, EN 12312-529, EN 12312-530, EN 12312-531, EN 12312-532, EN 12312-533, EN 12312-534, EN 12312-535, EN 12312-536, EN 12312-537, EN 12312-538, EN 12312-539, EN 12312-540, EN 12312-541, EN 12312-542, EN 12312-543, EN 12312-544, EN 12312-545, EN 12312-546, EN 12312-547, EN 12312-548, EN 12312-549, EN 12312-550, EN 12312-551, EN 12312-552, EN 12312-553, EN 12312-554, EN 12312-555, EN 12312-556, EN 12312-557, EN 12312-558, EN 12312-559, EN 12312-560, EN 12312-561, EN 12312-562, EN 12312-563, EN 12312-564, EN 12312-565, EN 12312-566, EN 12312-567, EN 12312-568, EN 12312-569, EN 12312-570, EN 12312-571, EN 12312-572, EN 12312-573, EN 12312-574, EN 12312-575, EN 12312-576, EN 12312-577, EN 12312-578, EN 12312-579, EN 12312-580, EN 12312-581, EN 12312-582, EN 12312-583, EN 12312-584, EN 12312-585, EN 12312-586, EN 12312-587, EN 12312-588, EN 12312-589, EN 12312-590, EN 12312-591, EN 12312-592, EN 12312-593, EN 12312-594, EN 12312-595, EN 12312-596, EN 12312-597, EN 12312-598, EN 12312-599, EN 12312-600, EN 12312-601, EN 12312-602, EN 12312-603, EN 12312-604, EN 12312-605, EN 12312-606, EN 12312-607, EN 12312-608, EN 12312-609, EN 12312-610, EN 12312-611, EN 12312-612, EN 12312-613, EN 12312-614, EN 12312-615, EN 12312-616, EN 12312-617, EN 12312-618, EN 12312-619, EN 12312-620, EN 12312-621, EN 12312-622, EN 12312-623, EN 12312-624, EN 12312-625, EN 12312-626, EN 12312-627, EN 12312-628, EN 12312-629, EN 12312-630, EN 12312-631, EN 12312-632, EN 12312-633, EN 12312-634, EN 12312-635, EN 12312-636, EN 12312-637, EN 12312-638, EN 12312-639, EN 12312-640, EN 12312-641, EN 12312-642, EN 12312-643, EN 12312-644, EN 12312-645, EN 12312-646, EN 12312-647, EN 12312-648, EN 12312-649, EN 12312-650, EN 12312-651, EN 12312-652, EN 12312-653, EN 12312-654, EN 12312-655, EN 12312-656, EN 12312-657, EN 12312-658, EN 12312-659, EN 12312-660, EN 12312-661, EN 12312-662, EN 12312-663, EN 12312-664, EN 12312-665, EN 12312-666, EN 12312-667, EN 12312-668, EN 12312-669, EN 12312-670, EN 12312-671, EN 12312-672, EN 12312-673, EN 12312-674, EN 12312-675, EN 12312-676, EN 12312-677, EN 12312-678, EN 12312-679, EN 12312-680, EN 12312-681, EN 12312-682, EN 12312-683, EN 12312-684, EN 12312-685, EN 12312-686, EN 12312-687, EN 12312-688, EN 12312-689, EN 12312-690, EN 12312-691, EN 12312-692, EN 12312-693, EN 12312-694, EN 12312-695, EN 12312-696, EN 12312-697, EN 12312-698, EN 12312-699, EN 12312-700, EN 12312-701, EN 12312-702, EN 12312-703, EN 12312-704, EN 12312-705, EN 12312-706, EN 12312-707, EN 12312-708, EN 12312-709, EN 12312-710, EN 12312-711, EN 12312-712, EN 12312-713, EN 12312-714, EN 12312-715, EN 12312-716, EN 12312-717, EN 12312-718, EN 12312-719, EN 12312-720, EN 12312-721, EN 12312-722, EN 12312-723, EN 12312-724, EN 12312-725, EN 12312-726, EN 12312-727, EN 12312-728, EN 12312-729, EN 12312-730, EN 12312-731, EN 12312-732, EN 12312-733, EN 12312-734, EN 12312-735, EN 12312-736, EN 12312-737, EN 12312-738, EN 12312-739, EN 12312-740, EN 12312-741, EN 12312-742, EN 12312-743, EN 12312-744, EN 12312-745, EN 12312-746, EN 12312-747, EN 12312-748, EN 12312-749, EN 12312-750, EN 12312-751, EN 12312-752, EN 12312-753, EN 12312-754, EN 12312-755, EN 12312-756, EN 12312-757, EN 12312-758, EN 12312-759, EN 12312-760, EN 12312-761, EN 12312-762, EN 12312-763, EN 12312-764, EN 12312-765, EN 12312-766, EN 12312-767, EN 12312-768, EN 12312-769, EN 12312-770, EN 12312-771, EN 12312-772, EN 12312-773, EN 12312-774, EN 12312-775, EN 12312-776, EN 12312-777, EN 12312-778, EN 12312-779, EN 12312-780, EN 12312-781, EN 12312-782, EN 12312-783, EN 12312-784, EN 12312-785, EN 12312-786, EN 12312-787, EN 12312-788, EN 12312-789, EN 12312-790, EN 12312-791, EN 12312-792, EN 12312-793, EN 12312-794, EN 12312-795, EN 12312-796, EN 12312-797, EN 12312-798, EN 12312-799, EN 12312-800, EN 12312-801, EN 12312-802, EN 12312-803, EN 12312-804, EN 12312-805, EN 12312-806, EN 12312-807, EN 12312-808, EN 12312-809, EN 12312-810, EN 12312-811, EN 12312-812, EN 12312-813, EN 12312-814, EN 12312-815, EN 12312-816, EN 12312-817, EN 12312-818, EN 12312-819, EN 12312-820, EN 12312-821, EN 12312-822, EN 12312-823, EN 12312-824, EN 12312-825, EN 12312-826, EN 12312-827, EN 12312-828, EN 12312-829, EN 12312-830, EN 12312-831, EN 12312-832, EN 12312-833, EN 12312-834, EN 12312-835, EN 12312-836, EN 12312-837, EN 12312-838, EN 12312-839, EN 12312-840, EN 12312-841, EN 12312-842, EN 12312-843, EN 12312-844, EN 12312-845, EN 12312-846, EN 12312-847, EN 12312-848, EN 12312-849, EN 12312-850, EN 12312-851, EN 12312-852, EN 12312-853, EN 12312-854, EN 12312-855, EN 12312-856, EN 12312-857, EN 12312-858, EN 12312-859, EN 12312-860, EN 12312-861, EN 12312-862, EN 12312-863, EN 12312-8

Key Safety Features & Benefits

- **Permanent Lock Dispose Button**
Once activated, blade cannot be re-exposed
- **Retractable Safety Sheath**
Covers blade before, during passing, and after use
- **One-Handed, Ambidextrous Design**
Use, passing, and disposal can be performed single-handedly by right- and left-handed users.
- **Audible Lock Confirmation**
Click confirms secure positioning
- **Premium Stainless Steel Blade**
Consistent sharpness and cutting performance
- **Colour-Coded Handle with Size Printed**
Fast identification in clinical settings

KIATO Plus® safety scalpels are available in commonly used surgical blade sizes to suit a wide range of procedures.

FITMENT 3


Size	Reorder no
10	12910
11	12911
11P	12931
12	12912
12D	12925
13	12913
15	12915
15C	12926

FITMENT 4

Size	Reorder no
18	12927
19	12919
20	12920
21	12921
22	12922
23	12923
24	12924
25	12928



Patent No: 7,097,087

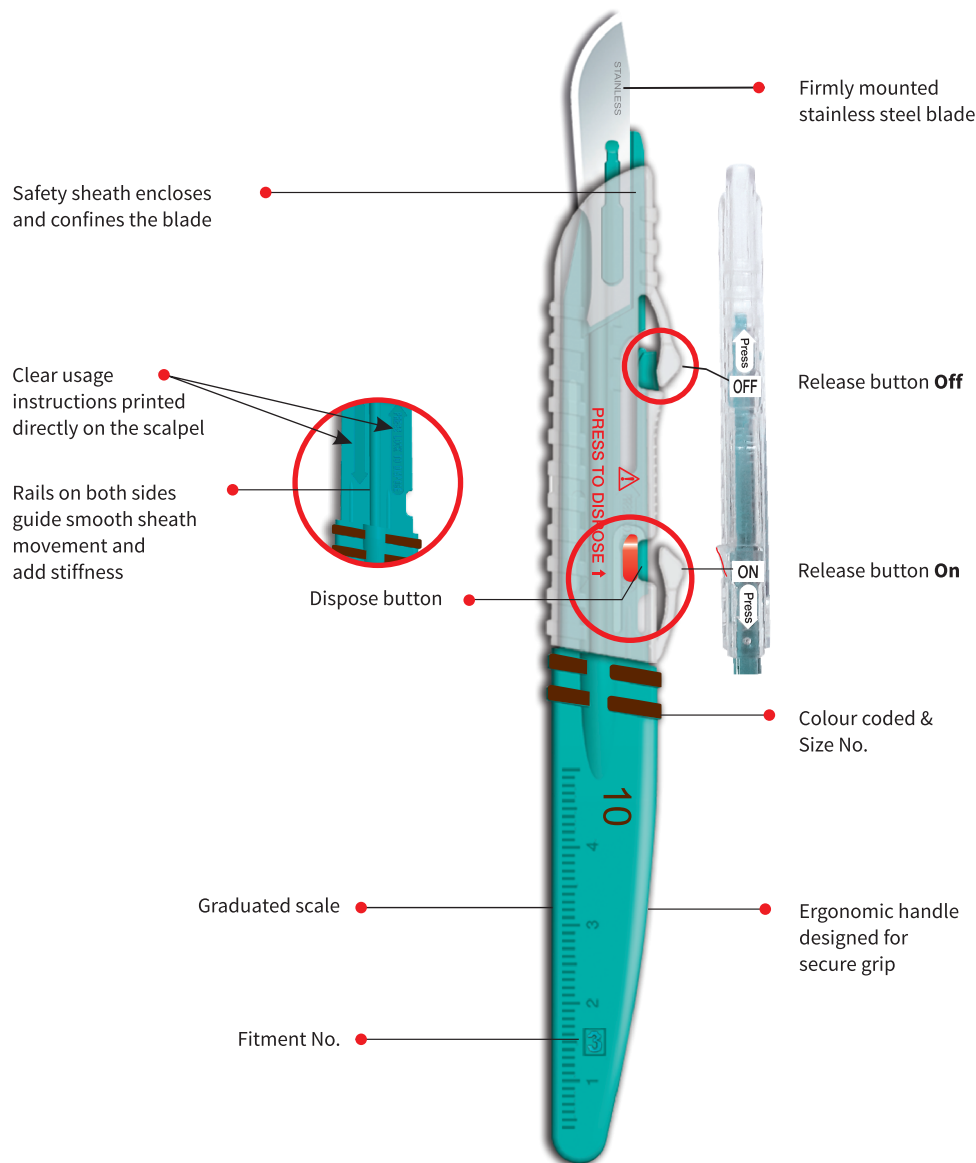

The smart choice for sharp blades

Aditya Dispomed Products Pvt. Ltd.
Plot No. 19 Sector 6 IMT Manesar
Gurgaon Haryana 122050 India
Tel. +91 124 47 64 904

sales@adityadisposed.com
www.adityadisposed.com

EU REP
MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany

CH REP
MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

Regulatory & Compliance

- CE marked medical device
- Classified as MD under EU MDR
- Designed to support compliance with EU Directive 2010/32/EU



Scan to view detailed product information and a usage demonstration.